

PMT® Halo System NEO-CERV® 1233

PMT® Corporation is an internationally certified (ISO 13485: 2003) distributor of medical products and devices. Our products are distributed in 36 different countries world-wide.

PMT® Corporation provides only FDA approved medical products and devices to its customers and consumers.

PMT Corporation, 1500 Park Road, Chanhassen, MN USA 55317, 1-800-MANKIND, 952-470-0866, Fax: 952-470-0865, Email: sales@pmtcorp.com



**PMT®** CORPORATION  
*Quality, Innovation and Customer Driven*

**PMT®**  
**Halo System**  
**NEO-CERV® 1233**



The gold standard in ambulatory fixation providing optimum support and patient comfort.

## Halo System NEO-CERV® 1233

This open back traction halo system comes complete with graphite composite open back ring, traction bail, head pins, positioning pads, Extended Range (ER) superstructure, vest, liner, and wrenches.



## Authorized Distributor

Health Group Industrias de México S.A. de C.V.  
Prol. Av. Vallarta 6245 Cd. Granja, Zapopan, Jalisco, México  
Tels.: 01(33) 3777-4000, 3777-4001, Fax: 01(33) 3777-4002,  
[licitaciones@ortiz.biz](mailto:licitaciones@ortiz.biz)  
Web: [www.ortiz.biz](http://www.ortiz.biz)

## UNPACKING YOUR HALO

The following components will be included with your PMT Halo System:

- A. (1) Halo Ring
- B. (3) Positioning Pins
- C. (5) Head Pins (Standard Ring)
  - C.1 (2) Spring Loaded Head Pins and
  - C.2 (2) Solid Head Pins (Traction Ring)
- D. (1) Vest with Liner
- E. (4) Graphite Rods
- F. (2) Headblocks
- G. (1) Small Screwdriver (standard superstructure only)
- H. (1) 7/6"-o" Combination Wrench for Locking Head Pins
- I. (2) Allen Wrenches (1 Large and 1 Small)
 


NOTE: For emergency use only. PMT recommends a torque driver for tightening head pins and superstructure.
- J. (1) Traction Bail Assembly
 

NOTE: Depending on the circumstances you may decide to apply traction prior to vest application. Apply the vest and put the patient into traction. The traction bail assembly easily attaches to the headblocks.
- K. (feature) Cardiac Crease
- L. PMT Torque driver, which includes a double sided bit.
- M. Weight Set with 8 pieces.

For your convenience certain items are provided sterile (unless otherwise indicated) double packaged Tyvek pouch.

### COMMENDED ADDITIONAL SUPPLIES FOR APPLICATION

- \*10 cc syringe
- \*22 Ga. X 1 ° inch needle
- \*Local anesthetic
- \*Sterile gloves
- \*Alcohol swabs
- \*Surgical prep
- 4x4 pads, scissors and razor

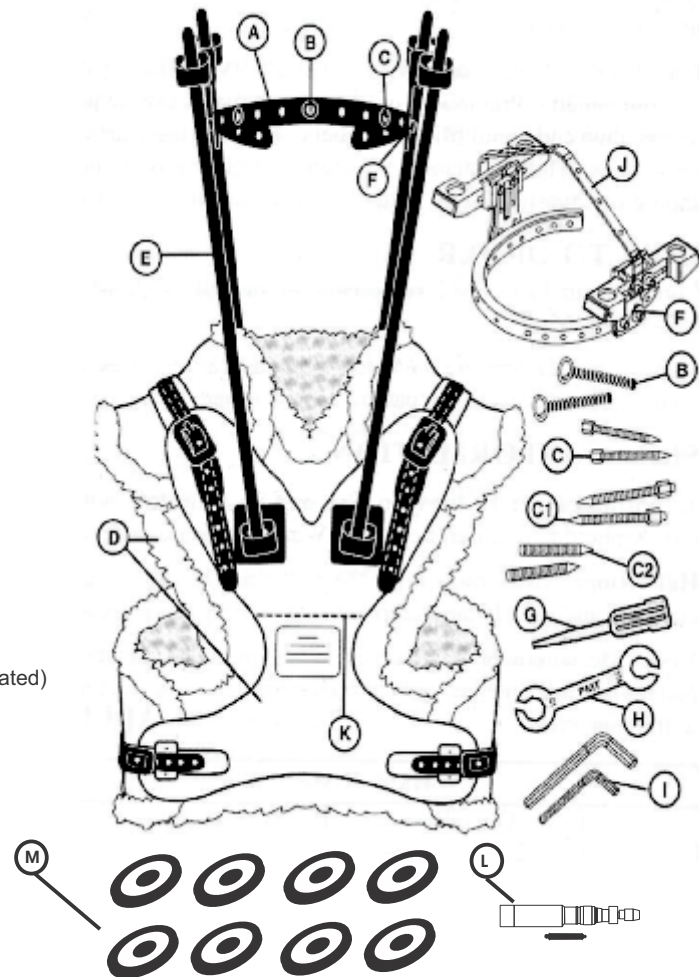

## BENEFITS

- 1.5/3T MRI Conditional plus CT compatibility
- Adjustable headblocks can independently adjust 3 ways
- Displaced buckle for clear lateral x-rays
- Shorter, tapered vest design to accommodate pendulous abdomen
- Soft, flexible material at shoulder for increased anterior/posterior compression
- Nylon ball joints for quick and easy fitting

## ACCESORIES

- Traction Tongs carbon graphite (titanium lock-nuts/S-hook) Model #1210
- Surgical Bed Adaptors (anterior/posterior) Model #1201-65
- Head Spoon Model #1201-16
- Skate Assembly (includes head spoon) Model #1201-12



Skate Assembly  
(includes head spoon)  
Model #1201-12



Surgical Bed  
Adaptors  
(anterior/posterior)  
Model #1201-65



Traction Tongs carbon graphite  
(titanium lock-nuts/S-hook)  
Model #1210

## ADULT VEST SIZES

ADULT VEST SIZES	(#1) Xy phod Circunference	(#2) Waist Circunference	(#3) Shoulder to xyphoid	(#4) Shoulder to illiac chest	(#5) Neck opening
Small	26"- 33" (66cm-84cm)	28"- 34" (71cm-86cm)	11.5" (29cm)	18.5" (42cm)	6.5" (17cm)
Medium	33"- 40" (84cm-102cm)	32"- 38" (81cm-97cm)	13" (33cm)	18" (46cm)	6.5" (17cm)
Large	40"- 46" (102cm-117cm)	36"- 42" (91cm-107cm)	14" (36cm)	19.5" (50cm)	6.5" (17cm)
X Large	46"- 50" (117cm-127cm)	40"- 46" (102cm-117cm)	17" (43cm)	22.5" (57cm)	8" (20cm)
XX Large	48"- 54" (122cm-137cm)	46"- 52" (117cm-132cm)	19" (48cm)	25" (64cm)	8.25" (21cm)